

## ED LABELING SYSTEM


### IV Tubing Labels

Scan manufacturer barcode




### Compounded Drugs

Scan QR code on laminated sheet or select drug from touch screen

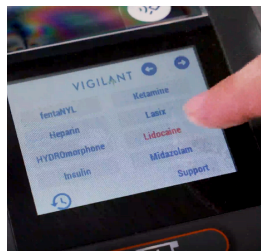


### Adult Syringe Labels

Select kit or individual drug from touch screen

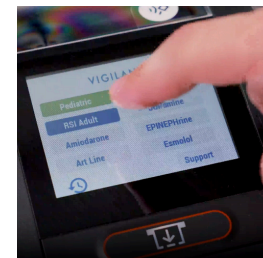


### Pediatric Syringe Labels

Select Pediatric on touch screen, select weight, select kit or individual drug

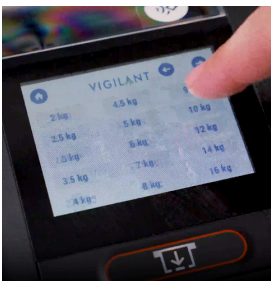

### Backtiming Labels

Select clock icon on touch screen, enter # of hours to subtract prior to printing label

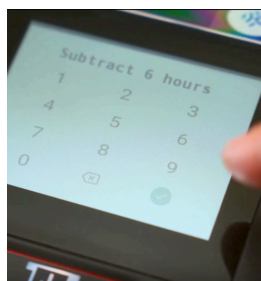

### Training Video

Scan the QR code using your phone to watch the training video



For library updates, technical concerns or support, please send us an email at [support@vigilantsoftware.io](mailto:support@vigilantsoftware.io) or call (469) 608-1930.