


IV TUBE LABELING SYSTEM

Scan the drug barcode.

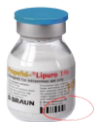

Once scanned, the printer will print the label.

Compounded Drugs

Press the category of the drug, then choose your medication

(Printer does not recognize compound drug barcodes)



Backtiming Labels

Prior to printing label, press "Exp 96" in bottom left corner, press clock icon, enter # of hours to subtract from current time



Non-drug labels

Press "Misc/Others" and choose the non-med label option.



If not found

Press the "Misc/Others" then choose one of the "Blank Label" options. Print, then write in the drug information



Training Video

Scan the QR code using your phone to watch the training video



For library updates, technical concerns or support, please send us an email at support@vigilantsoftware.io or call (469) 608-1930.